

DENTAL LIGHT-CURED RESTORATIVE COMPOSITE  
**CLEARFIL MAJESTY™ ES-2**

001 S3010-EU



DENTAL LIGHT-CURED RESTORATIVE COMPOSITE  
**CLEARFIL MAJESTY™ ES-2**

CE 0197 See Instructions for Use  
Kuraray Noritake Dental Inc.  
1621 Sakazu, Kurashiki,  
Okayama 710-0801, Japan

## Shade Description Chart

### Shade description and concept

Layering method	Shade category	Shade																
		Lightness/Value ← high → low																
Single-shade	Classic	XW	W	B1	A1	B2	D2	A2	C1	C2	A3	D3	B3	A3.5	B4	C3	A4	C4
Dual-shade	Premium	Enamel	XWE	WE	A1E		A2E		A3E		A3.5E		A4E					
		Dentin	XWD	WD	A1D		A2D		A3D		A3.5D		A4D					

※A, B, C & D shades represent VITA Classical shades. XW stands for Extra White and W stands for White.

- When B1 to C4 (15 VITA Classical shades) are arranged according to the lightness and divided into five groups, 'A' shades represent the center of each group.
- In the Premium system, use the corresponding combination of Enamel and Dentin shades.
- Combination of each Premium shade (A shades) covers 3 VITA shades. For example, the dual-layer of A1E and A1D covers B1, A1 and B2.

### Other shades

Shade category	Shade
Classic	A6
Translucent	Clear, Amber, Blue, Gray

- A6 has the lowest lightness in Classic shades.
- Translucent shades can be used for fine characterization of the restoration.

VITA approval only applies to VITA Classical shades and does not represent XW, W, A6 and Translucent shade categories.

"CLEARFIL" and "CLEARFIL MAJESTY" are trademarks of KURARAY CO., LTD.

④ "VITA" is a trademark of VITA Zahnfabrik, Bad Sackingen, Germany.

07/2012

# TECHNIQUE GUIDE

## Lightness/Value based shade taking with unique CLEARFIL MAJESTY ES-2 Shade guide

(For most restorations, 'VITA Classical shade guide' also can be used.)

### Option of single (Classic) and dual (Premium) shade systems

**Classic : Single-shade technique** by VITA Approved Shades  
incorporating Kuraray light diffusion technology

**Premium : Dual-shade technique** by VITA Approved Shade Concept

**Biomimetic layering system**

**Dentin replaced with a Dentin shade and Enamel with Enamel shade**

①

# TECHNIQUE GUIDE

DENTAL LIGHT-CURED RESTORATIVE COMPOSITE  
**CLEARFIL MAJESTY™ ES-2**

## Single-shade technique (Classic system) 'VITA Approved Shades'


### Shade-taking

#### Step 1:

Select the closest lightness/value tab from 6 exposed tab options of the Shade guide.


#### Step 2:

Pull out the chosen tab from the holder and select the corresponding shade.

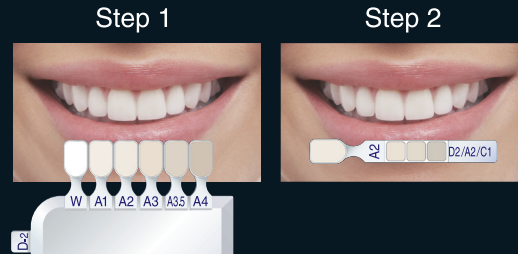

### Restoration

Classic shade

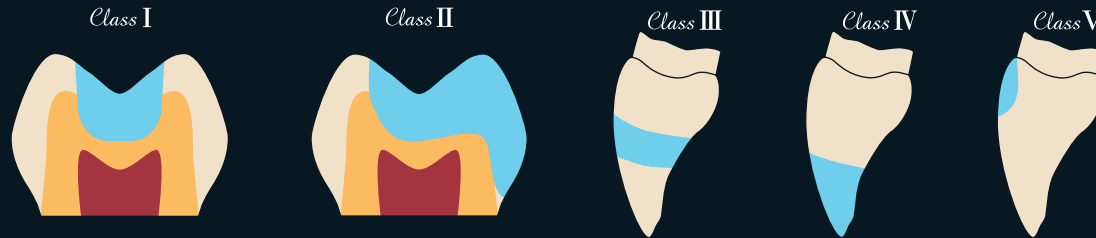

## Dual-shade Technique (Premium system) 'VITA Approved Shade Concept'

### Shade-taking

Select the closest lightness/value tab from 6 exposed tab options of the Shade guide.

For teeth lighter than W (white) shade, pull-out the tab for XW (extra-white) shade taking.

### Restoration

- Select the corresponding pastes of Premium shades (Enamel/Dentin). For example, when A1 tab is chosen, then select A1E and A1D shades.
- Replace dentin space with a respective Dentin shade, and enamel space with a respective Enamel shade.



Enamel shade    Dentin shade

