


SOFRELINER TOUGH

Intraoral Lining Method

Standard Handling Time

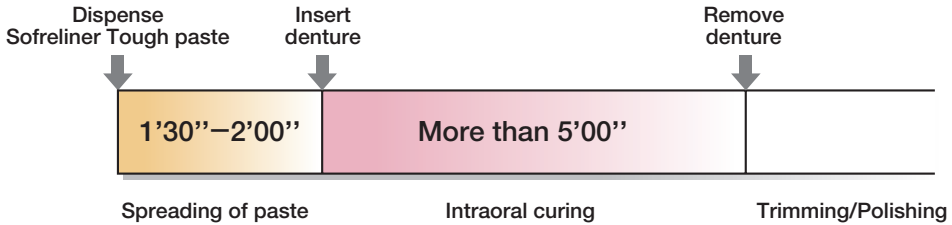

STANDARD PROCEDURE GUIDE

- Examine mouth**
- Examine denture**
- Mark a borderline on the denture using Marking Pen**
- Remove denture lining surface with round end tapered carbide bur**
- Bevel the borders of the denture to roll over the red marked borders**
- Rinse and dry denture surface**
- Apply Primer on the denture lining area, and then **DRY it sufficiently****
- Apply Sofreliner Tough paste evenly across all surfaces where Primer was applied**
- After intraoral curing, remove the denture from mouth and carefully trim excess paste with a sharp scalpel**
- Use **SHAPE ADJUSTMENT POINT (White)** to smooth the surface and adjust the shape**
- Use **FINISHING POINT (Brown)** to polish and erase red borderline.**
- Relining completed**

Notes for cleaning the denture and soft relining material (for patient)


On soft relining

Wipe with a sponge or gauze under running water


On acrylic resin and teeth

Scrub with denture brush

* Denture cleaning agent is recommended to keep denture clean.

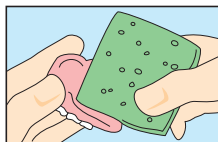

• **Sponge**

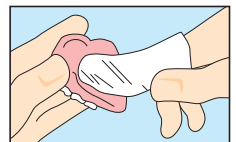


• **Gauze**

SOFRELINER TOUGH

Laboratory Lining Method

Standard Handling Time

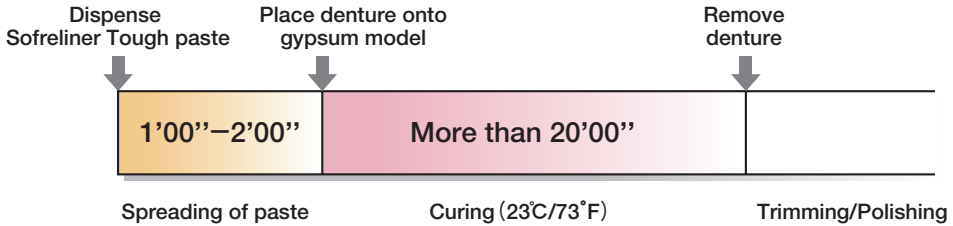

STANDARD PROCEDURE GUIDE

- Examine mouth
- Bite-seating impression
- Fabricate gypsum model and make a stone index of the teeth
- Mount to an articulator
- Make a relief in the gypsum model and apply resin separator on the whole area
- Mark a borderline on the denture using Marking Pen
- Remove resin with round bur for thickness of lining
- Remove denture lining surface with round end tapered carbide bur
- Bevel the borders of the denture to roll over the red marked borders
- Rinse and dry denture surface
- Apply Primer on the denture lining area, and then **DRY it sufficiently**
- Apply Sofreliner Tough paste evenly across all surfaces where Primer was applied
- Place the denture onto the gypsum model and allow the paste to cure
- After removing the denture, carefully trim excess paste with a sharp scalpel
- Use **SHAPE ADJUSTMENT POINT (White)** to smooth the surface and adjust the shape
- Use **FINISHING POINT (Brown)** to polish and erase red border line
- Relining completed

*Carefully read all instructions before using SOFRELINER TOUGH

Manufactured by Tokuyama Dental Corporation